



Chest U-Drape and Bar Drape

(79335/79375 depicted)



1. Orient body stamp on Chest U-Drape with patient and remove adhesive strip liner.



2. Center base of U-Drape at bottom of operative site.



3. Unfold drape, first laterally, then toward patient's feet, exposing tails of the U-Drape. Remove the adhesive strip liners from the U-Drape.




4. Position tails of U-Drape lateral to the operative site.



5. Orient the "Toward Incision" stamp on the Bar Drape with the patient. Remove adhesive strip liner from Bar Drape and center edge of Bar Drape at top of operative site.



6. Unfold laterally, then toward head to form an anesthesia screen, and extend armboard covers laterally. Insert surgical lines and tubing through the tube holders to complete draping.

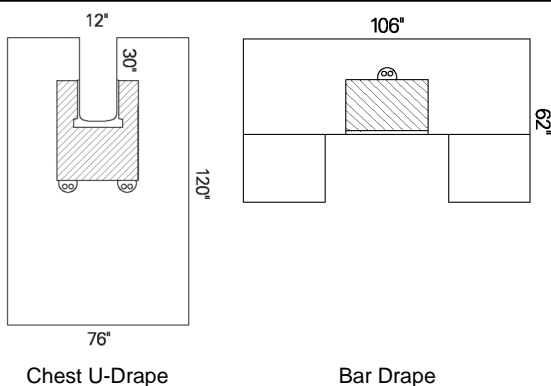

Features

Chest U-Drape

- 76" x 120"
- 12" x 30" split
- CONTROL* Fabric Reinforcement
- Tube Holders

Bar Drape

- Armboard Covers
- 106" x 62"
- CONTROL* Fabric Reinforcement
- Tube Holders

Draping Hints

- This drape is versatile and can be used for a number of different procedures.
- Placement of the Bar Drape allows you the flexibility to customize to the desired fenestration size.

Stock

Description

Items Per Case

79335 ♦

Chest U-Drape, Non-Sterile, Bulk Pack

22

79375 ♦

Bar Drape, Non-Sterile, Bulk Pack

28

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. *Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.

<See the following page for Pack Options>

Pack Stock # 88331		Description <i>U-BAR* Pack III - Chest</i>	Packs/Case 4
Component Stock #			
Sterile	Non-Sterile ♦	Contents:	
89618	79498	1 Back Table Cover, Zone-Reinforced, 44" x 90"	
95111	70111	1 ULTRA Surgical Gown, Large, in Overwrap with Towel	
95211	70211	2 ULTRA Fabric-Reinforced Surgical Gowns, Large, with Towels	
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"	
89731	79731	4 Utility Drapes, 26" x 15", with Tape	
		1 Suture Bag	
	79335	1 Chest U-Drape, 76" x 120", 12" x 30" Split <ul style="list-style-type: none"> • CONTROL* Fabric Reinforcement • Tube Holders 	
	79375	1 Bar Drape, 106" x 62", with Armboard Covers <ul style="list-style-type: none"> • CONTROL* Fabric Reinforcement • Tube Holders 	

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. *Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.