


Laparoscopy "T" Drape

(89119 depicted)



1. With Leggings and an Under Buttocks Drape in place, orient body stamp on drape with patient. Remove adhesive strip liners clockwise.

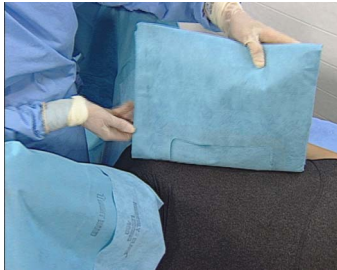

2. Position drape with bottom of fenestration just above pubic bone over abdominal incision site.



3. Unfold drape, first laterally and then, following arrows, bring drape down between legs to expose perineal fenestration. Firmly seal abdominal fenestration to patient's skin.




4. Unfold upper portion toward head to form an anesthesia screen.



5. Perineal flap helps to isolate the perineal area.



6. Insert surgical lines and tubing through the tube holders to complete draping.

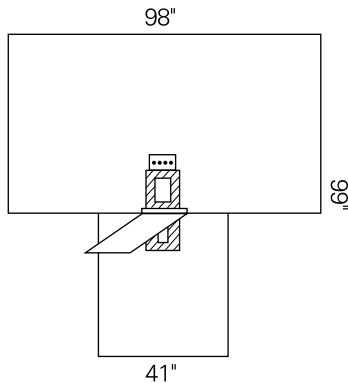

Features

- 98" x 41" x 99"
- 6" x 8" Abdominal Fenestration
- 3" x 6" Perineal Fenestration with 15" x 21" Flap
- CONTROL* PLUS Fabric Reinforcement
- Tube Holders

Stock #	Description	Items Per Case
89119	Laparoscopy "T" Drape, Sterile, 1 per package	19
79117 ♦	Laparoscopy "T" Drape, Non-Sterile, Bulk Pack	26

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

Pack Stock #
88088

Description
Laparoscopy Pack I

Packs/Case
10

Component Stock #

Sterile Non-Sterile ♦

Contents:

89618	79498	1 Back Table Cover, Zone-Reinforced, 44" x 90"
89701	79720	1 Absorbent Towel, 15" x 26"
89411	79412	1 Under Buttocks Drape, Plastic, 29½" x 44"
89731	79731	1 Utility Drape, 26" x 15", with Tape
89408	79408	2 Leggings, with 6" Cuffs
	79117	1 Laparoscopy "T" Drape, 98" x 41" x 99" <ul style="list-style-type: none">• 6" x 8" Abdominal Fenestration• 3" x 6" Perineal Fenestration, with 15" x 21" Flap• CONTROL* PLUS Fabric Reinforcement• Tube Holders

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. *Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.

<See the following page for Pack Options>